SENIOR PLUS SENIOR PLUS2

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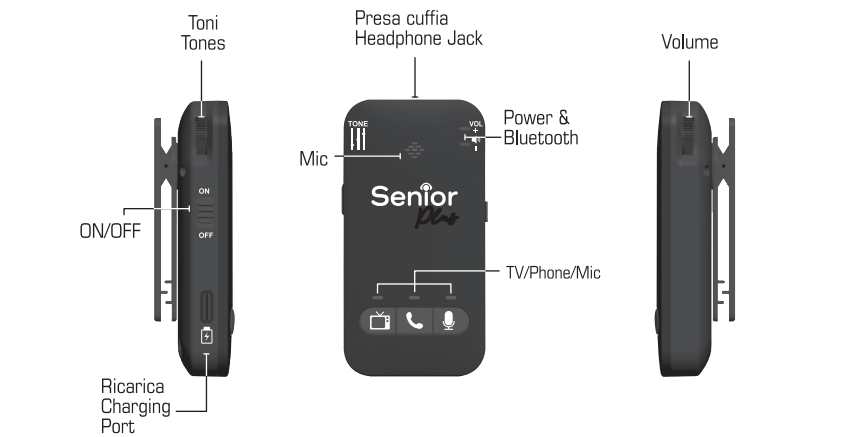
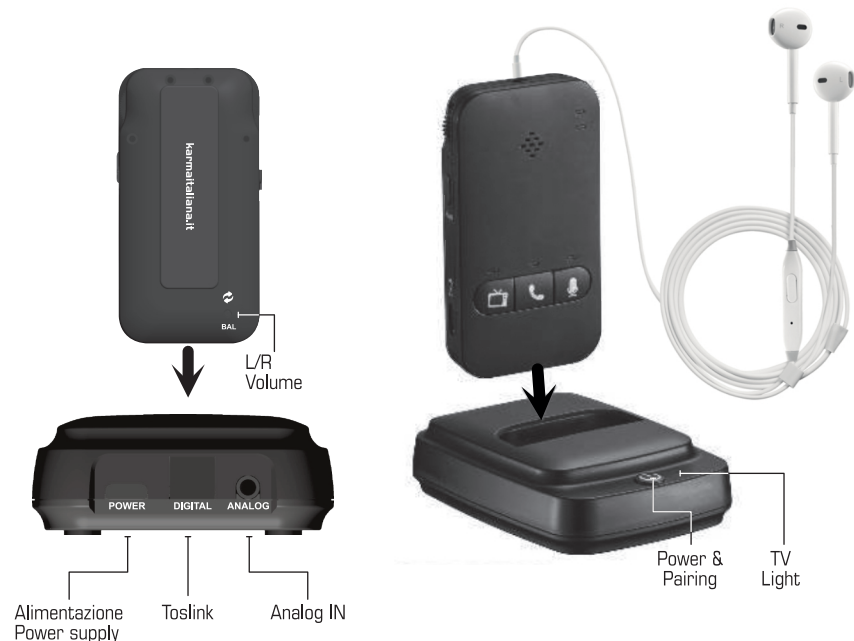
INTRODUCTION

We thank you for purchasing this extraordinary product specifically designed for old people or anyone with hearing problems. It connects to the TV or the audio source you want to listen to and thanks to the built-in Bluetooth receiver it notifies you of an incoming phone call by lowering the volume level and getting the phone ringtone to be heard through the headphones. Simply press a button to answer the call without even getting up from the sofa. But that's not all: this wonderful device features many other functions, such as volume and tone rotating control, sound balance between the right and left ear, ambient monitoring button and much else.... The package contains a pair of ordinary earphones, but you can connect your own headphones or purchase our specific model (HPS SEN), which offers highest quality and comfort of use.

This manual is referred to 2 products (SENIOR PLUS and SENIOR PLUS2), which differ from each other for the number of receivers in the packaging. Model SENIOR PLUS is intended for a single user, while model SENIOR PLUS2 allows two persons to listen to audio in their headphones at the same time, but independently. If you have bought model SENIOR PLUS2 in the packaging you will find some stickers to be applied on the receivers. In this way you will be able to distinguish the two receivers, if connected to two smartphones, in order to answer to the correct incoming calls.

CAUTIONS AND WARNINGS

- Read carefully the instructions contained in this manual.
- Do not open or take apart the device and don't make changes to it.
- Don't expose it to high temperatures (above 40°C), rain or humidity.
- Do not allow flammable liquid, water or metal objects to enter the product.
- Use the product, its controls and the accessories with due accuracy, avoiding falls, vibrations or impacts.
- In the event of operating problems, stop using the product immediately and contact the authorized technical service center.

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WARRANTY CONDITIONS
The fixture is covered by standard 12 or 24 months warranty according to the current regulation. The warranty is no longer valid if the fixture is disassembled, modified or improperly used. For detailed warranty conditions please refer to our website www.karmaitaliana.it.

BATTERY WARNINGS
The product is powered by a rechargeable lithium battery. Do not remove, open or modify the battery. To avoid the risk of fire or explosion, do not expose the battery to flames, heat, humidity and avoid shocks. In the event of battery leaking, if the liquid comes into contact with the skin or eyes wash the affected area with copious water and consult a doctor if necessary.

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RECEIVER RECHARGE

Connect the transmitter base to a domestic power socket by means of the included USB cable and place the receiver on the base. Around 2 hours are necessary to complete the recharge. Each time you hear the low battery warning, recharge immediately. A full discharge could ruin the battery. The headphones will be automatically be recharged if left on the base when not in use. If you do not use the headphones for long periods, turn off the base and disconnect it from the power outlet. We suggest not placing the transmitter base near Wi-Fi devices that could generate interferences. In the 2-receiver version, as only one charging base is available, you will have to alternate recharge of the two receivers, or recharge one of them directly via a USB Type-C cable thanks to the side charging socket.

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
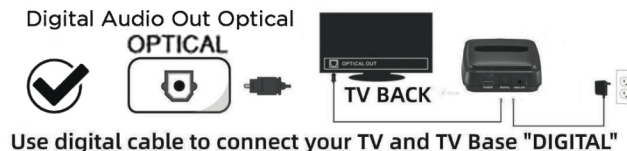
① Digital Smart TV


Use digital cable to connect your TV and TV Base "DIGITAL"

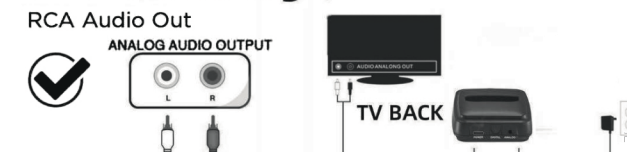② Analog TV

Use RCA Cable to Connect Your TV and TV Base "ANALOG"

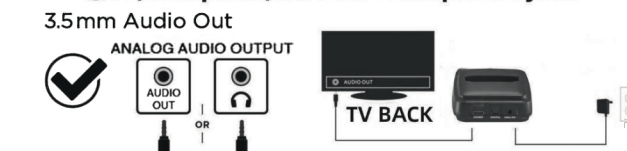③ TV, Computer, etc with headphone jack

Use 3.5mm Cable to Connect Your TV and TV Base "ANALOG"

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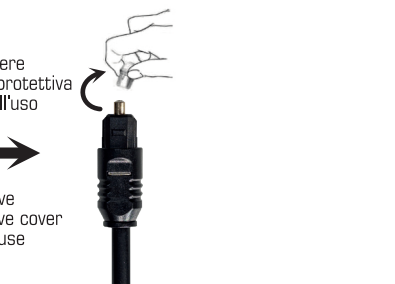
WARNING

If you mean to use the optical cable check that there are no protection covers, before inserting it into the optical socket. You would risk to damage the connectors. Remove the protections if present.

If you use the connection via optical fibre cable, on some TVs it could be necessary to enter the audio menu to activate the PCM format and avoid that the signal picked up by the base is disturbed or unreadable.

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The screen shown in the picture could be different according to the user's TV model.

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
Hereby, KARMA ITALIANA SRL declares that the items wireless headphones SENIOR PLUS/ SENIOR PLUS 2 are in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address:

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LED POSITION	COLOR	STATE	INDICATIONS
BASE (TRANSMITTER)	BLUE	STABLE	BASE AND RECEIVER CONNECTED
	BLUE	BLINKS	BASE IS CONNECTING TO RECEIVER
	BLUE	QUICK	BASE IS CONNECTING TO NEW RECEIVER
	OFF	OFF	BASE OFF OR IN STANDBY
RECEIVER CHARGE LED	RED	STABLE	RECHARGING
	RED	BLINKS	BATTERY LOW - NEED TO BE RECHARGED
	OFF	OFF	RECHARGING COMPLETED
RECEIVER BLUETOOTH LED	BLUE	BLINKS	RECEIVER IS CONNECTING TO PHONE
	OFF	OFF	RECEIVER CONNECTED TO PHONE
RECEIVER TV BUTTON	BLUE	STABLE	RECEIVER CONNECTED TO BASE
	BLUE	BLINKS	RECEIVER IS CONNECTING TO BASE
	BLUE	QUICK	BASE AND RECEIVER ARE PAIRING
RECEIVER PHONE BUTTON	BLUE	STABLE	RECEIVER CONNECTED TO PHONE
	BLUE	BLINKS	RECEIVER IS CONNECTING TO PHONE
RECEIVER MICROPHONE BUTTON	BLUE	STABLE	AMBIENT MICROPHONE ACTIVATED

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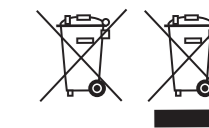Once you have finished with this function, lower the volume again before pressing the TV button again. Then adjust the listening

EXCLUSIVE CONTROLS

This fixture is also conceived to be used by elderly people who often have hearing problems. Thanks to the very high listening volume and the incoming calls notification it is an exclusive product. But that's not all: it also offers the possibility of adjusting the tone of the audio you are listening to (top left knob) and balancing the sound as desired between the right and left ear, by turning the screw on the receiver's back with the included screwdriver.

AFTER USE

Turn off the receiver, by means of the side button, and place it on the base, where it will start charging thanks to the magnetic pins. If you have a further receiver, you can also recharge it directly via USB cable, thanks to the side charging port.

The base can be turned off by holding the power button for 3 sec. In any case, it will automatically turn off after 10 minutes without signal. When turned on again it will take up to 10 seconds to connect to the receivers.

1. Turn on the receiver and press the "phone" button. The corresponding led will flash indicating that the receiver is in pairing mode.
2. Turn on your smartphone and enter the Bluetooth menu.
3. Search for the device named SENIOR PLUS and confirm the connection on the phone.
4. Done! The receiver has saved the phone and will automatically connect without requiring again this procedure.

SMARTPHONE DISCONNECTION

If you want to disconnect the phone from the receiver (to connect for example a new smartphone) you can operate as follows:

1. Turn on the receiver
2. Press and hold for 3 sec the receiver "MIC" button. The connection to the smartphones will be reset and the phone will be now disconnected.
3. Proceed with the pairing of the new smartphones as previously explained.

SPEED DIAL CALL

It is possible to save a phone number and then call it quickly from the receiver. Have the desired number call you and during the conversation hold down the "Phone" button for 3 seconds. You will hear a message confirming the number was entered into memory.

To call the number you have saved just press and hold the "Phone" button and to start dialling the number.

To change the speed dial number just repeat the above procedure.

AUDIO AMPLIFIER

SENIOR PLUS features a particular function, a sort of ambient microphone, capable of amplifying the surrounding sounds and giving the possibility of speaking face to face with other people even if you have hearing problems. Before activating the function, lower the volume to avoid sudden noises. Press the "Mic" button on the receiver and adjust the listening volume as desired.

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This solution compels you to disconnect the TV's headphone jack every time you finish using the headphones. For this reason we recommend using the optical cable.

RECEIVER/ BASE PAIRING

The base and the receiver included in packaging are already paired, then ready for use without having to do anything. If it is necessary to reset the system or if you need to buy an additional receiver unit, you will have to proceed in this way:

1. Turn on the new receiver
2. Press and hold for about 3 sec the power button on the base until it turns off.
3. Press and hold for about 6 sec the power button on the base until you see the led flashing quickly. Now the base is in pairing mode.
4. Press and hold for about 3 sec the "TV" button on the receiver. The led will flash quickly and in the headphones you will hear the ongoing pairing message.
5. After a few instants the base and the receiver will automatically connect, you will hear in the headphones a message indicating successful connection and the leds will stop flashing.
6. It is potentially possible to connect unlimited receivers to the transmitter base. In this way many persons will be able to listen to audio without interferences. It is enough to stay within the base transmission range.

PAIRING WITH SMARTPHONE

The SENIOR PLUS is the only TV headset that can notify you of an incoming phone call, allowing you to answer the call without even touching the phone. It will also be possible to listen to music from the smartphone in the headphones, thanks to the connection of the smartphone to the receiver via bluetooth. The first time you have to pair the phone to your receiver (if you have bought the dual receiver version you can connect 2 smartphones to the 2 receivers). This is the procedure to follow for bluetooth pairing:

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If you still don't hear the TV audio in the headphones you can try to control the settings in the TV's audio menu. It is often necessary to enable the optical output.

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In the rare case that your TV is endowed only with coaxial or HDMI type audio connectors, you necessarily have to buy a converter to connect SENIOR PLUS or other wireless headphones with optical socket.

SOME SUGGESTIONS

To listen to audio only in the headphones you can press the remote control mute button, while the audio via optical socket will remain active. If you connect the transmitter to the TV via 3.5mm jack, the TV speakers are often automatically disabled.